

## Products

---

FSN Medical Display Monitors.

## Precautions

---

- Before cleaning, switch the display in stand-by position to prevent the control touch panel from being activated inadvertently by sweeping over the front filter. In stand-by position the touch panel cannot be activated by just sweeping over them. To switch the display on again, you must press the stand-by key again.
- Take care not to damage or scratch the front filter or LCD panel.
- Be careful with rings or other jewelry that can touch the front filter.
- Do not apply pressure on the front filter or LCD panel.
- Do not apply or spray liquid directly to the front filter, panel or cabinet as excess liquid may cause damage to internal electronics. Instead, apply the liquid to the cleaning cloth.
- Follow your hospital protocol for the handling of blood and body fluids.
- The display is not disinfected or packed in sterile environment.
- Follow your hospital protocol in case the display needs to be disinfected prior to installation.

## Front Filter

---

Proceed as follows:

1. Remove dust with a dry, lint-free, non-abrasive soft cotton cloth.
2. Remove fingerprints or grease using a lint-free, non-abrasive soft cotton cloth that is lightly moistened with plain water or a mild commercial glass cleaning product suited for coated glass surfaces.
3. Gently wipe dry with a dry cloth.

The following products are tested and approved:

- Misty Clear Lemon 10 Disinfectant
- Bohle glass cleaner
- Zep Heavy-duty glass & all surface cleaner
- Klear Screen
- Screen TFT (Kontakt Chemie)
- Incidin Foam (Ecolab)
- Microzid
- Mild detergent
- Isopropyl alcohol with concentration < 5%
- Household bleach (generic sodium hypochlorite, solutions of 5.25% sodium hypochlorite diluted with water between 1:10 and 1:100)

Do NOT use:

- Alcohol/solvents at higher concentration > 5%
- Strong alkalis lye, strong solvents
- Acid
- Detergents with fluoride
- Detergents with ammonia
- Detergents with abrasives
- Steel wool
- Sponge with abrasives
- Steel blades
- Cloth with steel thread

## Cabinet

---

Proceed as follows:

1. Clean the cabinet using a soft cotton cloth, lightly moistened with a recognized cleaning product for medical equipment.
2. Repeat with water only.
3. Wipe dry with a dry cloth.
4. Cabinet has been tested for resistance to the following products:

- Virex Ready-to-use Disinfectant Cleaner
- Misty Clear Lemon 10 Disinfectant
- Misty Multi-Purpose Disinfectant Cleaner
- Misty Multi-Purpose Disinfectant Cleaner II
- Bohle glass cleaner
- Zep Heavy-duty glass & all surface cleaner
- Klear Screen
- Screen TFT (Kontakt Chemie)
- Incidin Foam (Ecolab)
- Microzid
- Mild detergent
- Isopropyl alcohol with concentration < 5%
- Household bleach (generic sodium hypochlorite, solutions of 5.25% sodium hypochlorite diluted with water between 1:10 and 1:100)
- Precise Hospital Foam Cleaner Disinfectant